

Sterilization Procedure for Wells Johnson Cannulas

Each cannula needs to be carefully scrubbed with a cannula brush, dried, packaged and treated in an autoclave at 265 degrees with 25 psi for 15 minutes or more. Cannulas must be sterilized before each use.

IMPORTANT NOTICE:

This is if you have a 2-piece cannula with a black Delrin handle. The melting point is between 329 and 347 degrees. We recommend to autoclave 265 degrees with 25 psi for 15 minutes or more. To ensure effective sterilization the handles should be scrubbed and dried before being sterilized. NEVER autoclave the handle with the cannula tip still attached to the handle. The two different materials in the autoclave at the same time can cause damage.

Should you have further concerns or questions, please contact the Wells Johnson Company at 1-800-528-1597.

Thank you

your source for innovation and new technology