

## CLEANING RECOMMENDATIONS

### INFUSION SYSTEMS AND ASPIRATORS

➤ **Infusion Systems and Aspirators-all models:**

Using a hospital grade cleaner and towel, clean a small and inconspicuous area of the device first to make sure the cleaner you are using is not damaging the device. If there is visible damage, you may need to use a different cleaner or dilute the one used.

Once you've verified the cleaner is effective without causing damage, wipe down the top, all sides and back of unit as needed.

*\*Some devices have a touch screen control panel. To clean the touch screen, dampen a towel with a mild detergent and water and gently wipe the screen until clean.*

Applicable to the following Wells Johnson Part #'s:

20-5000-00	20-5061-00
20-5001-00	20-5066-00
20-5007-00	20-6000-00
20-5010-00	20-6005-00
20-5026-00	90-6200-00
20-5028-00	90-6201-00